


# Correct application of barcode label to S-Monovette® with transparent plastic label

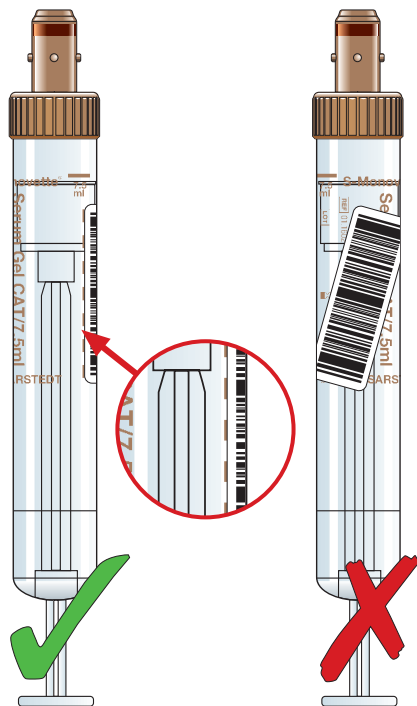

Right

Wrong


- Stick the barcode label **along the dashed line** on the plastic label on the S-Monovette®
- Do not stick the barcode label **at an angle or around** the tube

Sample containers with a plastic label are correctly labelled if:

- the barcode label is applied along the dashed line on the plastic label on the S-Monovette®.
- the contents can be seen clearly.
- it is possible to check the level of the contents. This requires the label to be applied below the graduation markings.
- the label does not prevent the screw cap from being removed.
- the tube and label do not jam or stick to the centrifuge.



# Correct application of barcode label to S-Monovette® with paper label

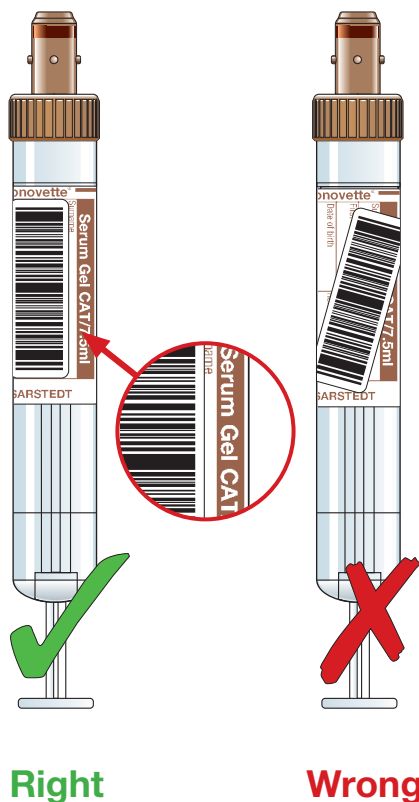

- Stick the barcode label precisely **over** the paper label that is on the S-Monovette®. The printed description of the preparation should be to the right of the barcode label.
- Do not stick the barcode label **next to** the paper label, as a clear view of the sample is required for visual inspections.

Sample containers with a paper label are correctly labelled if:

- the barcode label is applied precisely over the paper label that is on the S-Monovette®.
- the contents can be seen clearly.
- it is possible to check the level of the contents.
- the label does not prevent the screw cap from being removed.
- the tube and label do not jam or stick to the centrifuge.



SARSTEDT AG & Co. KG  
P.O. Box 12 20  
D-51582 Nümbrecht  
Phone: +49 2293 305 0  
Fax: +49 2293 305 3992  
export@sarstedt.com  
www.sarstedt.com