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Intended use

The Multi-Safe Disposal Boxes twin plus are waste containers to be used exclusively for the disposal of pointed and sharp objects, such as cannulas and scalpel blades for medical use.

Product description

The waste container is made of polypropylene. It consists of a box and a lid. Before filling the container, the lid must be attached and secured into place in order to ensure safe disposal. The waste container can be used only once.

Product handling

The size of the filling opening can be enlarged incrementally by breaking the two flaps prior to assembling the lid. Prior to filling, press the lid **1** firmly onto the box base **2** until it audibly and visibly clicks into place all the way around.

Prior to filling, twist the box onto the recommended adhesive adapter or wall holder to prevent the box from tipping over.

Ensure items for disposal are placed into the box only in a vertical direction without using force. Scalpels must be placed into the box with utmost care. Using too much force or filling the box with other items may result in items becoming jammed, damaging the walls or the base of the box.

For daily use, press the day lid **4** in slightly towards the opening, then the box can be opened again.

There is a fill line on the upper part of the boxes **3**. For safety reasons, do not fill the box above this line **3**.

Removal aids:

U-shaped recess **A** for removing sharps from Luer lock systems.





Star-shaped notched opening **B** for disassembling pens.

Recess with sections **C** for removing cannulas.

Final closure of the day lid **4**:

After filling the box (Caution: Do not fill above the fill line **3**), proceed as follows: Place the box on a stable surface and press the day lid **4** firmly into the filling opening until it audibly and visibly clicks into place.

The upper part of the box has a handle **5** that can be folded upwards by gently pulling on it. The box can now be easily carried to the next disposal site.

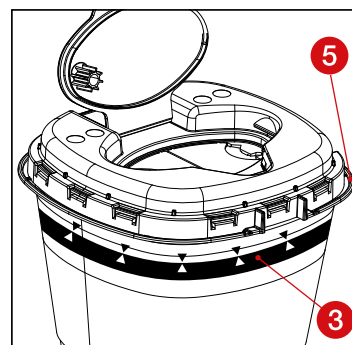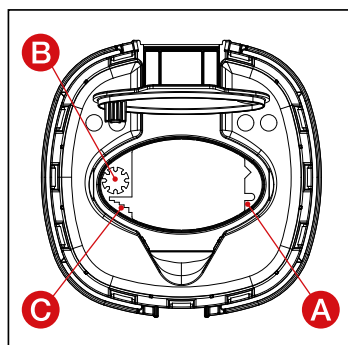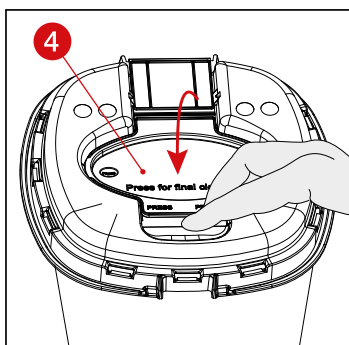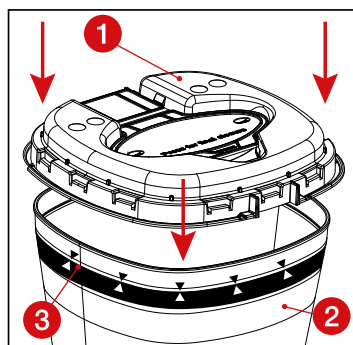Safety information and warnings

1. Use only boxes that are suitable for holding the items for disposal.
2. Keep the box out of the reach of children.
3. Ensure that the lid is attached and clicked into place before filling.
4. Do not use force to place items into the box.
5. Do not pour liquids into the box.
6. Do not fill the box above the fill line **3** (risk of injury!).
7. Do not reach into the box with your hand or in any other way (risk of injury!).
8. Do not throw, drop or shake the box.
9. Ensure that no objects are protruding from the opening before closing the box.
10. Do not press the box against your body (e.g. chest) for the final closure.
11. Do not squeeze the sides of the box together when full.
12. Use the handle **5**.
13. Ensure the lid **1** and the box **2** are securely closed before disposal.

Storage

Store the product at room temperature.

Disposal

The boxes must be disposed of in a suitable incinerator.

Key for symbols and labels:

Article number

Country of manufacture

Manufacturer

If reused: Risk of contamination

Follow the instructions for use

Keep away from sunlight

Store in a dry place

Do not use if package is damaged

Technical modifications reserved

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

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