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SARSTEDT Minivette® POCT

DE	Gebrauchshinweis – SARSTEDT Minivette® POCT	2 - 4
EN	Instructions for use – SARSTEDT Minivette® POCT	5 - 7
BG	Инструкции за употреба – SARSTEDT Minivette® POCT	8 - 10
CS	Návod k obsluze – SARSTEDT Minivette® POCT	11 - 13
DA	Brugsanvisning – SARSTEDT Minivette® POCT	14 - 16
EL	Οδηγίες χρήσεως – SARSTEDT Minivette® POCT	17 - 19
ES	Instrucciones de uso – SARSTEDT Minivette® POCT	20 - 22
ET	Kasutusjuhend – SARSTEDT Minivette® POCT	23 - 25
FR	Mode d'emploi – SARSTEDT Minivette® POCT	26 - 28
HR	Uputa za upotrebu – SARSTEDT Minivette® POCT	29 - 31
HU	Használati utasítás – SARSTEDT Minivette® POCT	32 - 34
IT	Istruzioni d'uso – SARSTEDT Minivette® POCT	35 - 37
KO	사용 설명서 – SARSTEDT Minivette® POCT	38 - 40
LT	Naudojimo instrukcijos – SARSTEDT Minivette® POCT	41 - 43
LV	Lietošanas norādes – SARSTEDT Minivette® POCT	44 - 46
NL	Gebruiksaanwijzing – SARSTEDT Minivette® POCT	47 - 49
NO	Bruksanvisning – SARSTEDT Minivette® POCT	50 - 52
PL	Instrukcja obsługi – próbówka Minivette® POCT SARSTEDT	53 - 55
PT	Instruções de utilização – SARSTEDT Minivette® POCT	56 - 58
RO	Instrucțiuni de utilizare – Minivette® POCT SARSTEDT	59 - 61
RU	Инструкция по применению – SARSTEDT Minivette® POCT	62 - 64
SK	Návod na Použitie – SARSTEDT Minivette® POCT	65 - 67
SL	Navodila za uporabo – Minivette® POCT SARSTEDT	68 - 70
SV	Bruksanvisning – SARSTEDT Minivette® POCT	71 - 73
TH	คำแนะนำในการใช้งาน – ระบบการเจาะเลือดสำหรับการทดสอบ ณ จุดดูแลผู้ป่วย Minivette® POCT ของ SARSTEDT	74 - 76
TR	Kullanım için talimatlar – SARSTEDT Minivette® POCT	77 - 79
ZH	使用说明 – SARSTEDT Minivette® POCT 采样管	80 - 82

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Intended use

The Minivette® POCT is used for capillary blood collection based on the end-to-end capillary principle. The Minivette® POCT is used for collecting and immediately dispensing a defined volume of capillary blood for point-of-care testing. The product is intended for use in a professional environment by qualified medical personnel.

Product description

The Minivette® POCT consists of a plastic capillary and a plunger, as well as various additives (preparations). A Minivette® POCT without additive is also available. The selection of the additive depends on the analytical test method. It is specified by the manufacturer of the test reagents and/or the analysis equipment, test cartridge or test strip on which the test is conducted.

Minivette® POCT colour codes:

Additive	Letter code	Plunger colour based on ISO 6710	Plunger colour based on BS 4851*
Neutral			
Minivette® POCT 10 µl neutral	Z	white	white
Minivette® POCT 20 µl neutral	Z	white	white
Minivette® POCT 50 µl neutral	Z	white	white
Minivette® POCT 100 µl neutral	Z	white	white
Minivette® POCT 200 µl neutral	Z	white	white
Lithium heparin tube			
Minivette® POCT 20 µl with Lithium Heparin	LH	green	orange
Minivette® POCT 50 µl with Lithium Heparin	LH	green	orange
Minivette® POCT 100 µl with Lithium Heparin	LH	green	orange
Minivette® POCT 200 µl with Lithium Heparin	LH	green	orange
EDTA tube			
Minivette® POCT 20 µl with K3 EDTA	K3E	violet	red
Minivette® POCT 50 µl with K3 EDTA	K3E	violet	red
Minivette® POCT 100 µl with K3 EDTA	K3E	violet	-
Minivette® POCT 200 µl with K3 EDTA	K3E	violet	-

*British Standard BS4851:1982 Specification for single use labelled medical specimen containers for haematology and biochemistry.

SARSTEDT Minivette® POCT neutral

The Minivette® POCT neutral contains no additive. The defined volume of native capillary blood can be used immediately after removal for point-of-care testing.

SARSTEDT Minivette® POCT Lithium Heparin

The Minivette® POCT Lithium Heparin is coated with the anticoagulant lithium heparin. The defined volume of anticoagulated capillary blood can be used immediately after collection for point-of-care testing.

SARSTEDT Minivette® POCT K3 EDTA

The Minivette® POCT K3 EDTA is coated with the anticoagulant K3 EDTA. The defined volume of anticoagulated capillary blood can be used immediately after collection for point-of-care testing.

Safety information and warnings

1. General precautions: Use gloves and other general personal protective equipment to protect yourself against blood and possible exposure to pathogens transmitted by blood.
2. Handle all biological specimens and sharp blood collection utensils (lancets) according to the guidelines and procedures in your facility. In case of direct contact with biological specimens or a needlestick injury, consult a doctor due to the risk of transmission of HIV, HCV, HBV or other infectious diseases. Use safety lancets. Always follow the safety guidelines and procedures of your facility.
3. Dispose of all sharp objects (lancets) for capillary blood collection in appropriate hazardous biological waste containers.
4. Underfilling the Minivette® POCT leads to an incorrect ratio of blood to preparation/additive and can result in false analysis results. (Overfilling can be ruled out with the Minivette® POCT because the blood flow in the Minivette® POCT is stopped by the filter).
5. The blood collected and processed with the Minivette® POCT is not intended to be re-injected into the human body.
6. Do not use Minivette® POCT after the expiry date. The expiry date of Minivette® POCT is the last day of the month and year as indicated.

Storage

Store the product at room temperature.

Specimen collection and handling

READ ALL OF THIS DOCUMENT BEFORE STARTING TO DRAW CAPILLARY BLOOD.

Material required but not supplied for capillary blood collection:

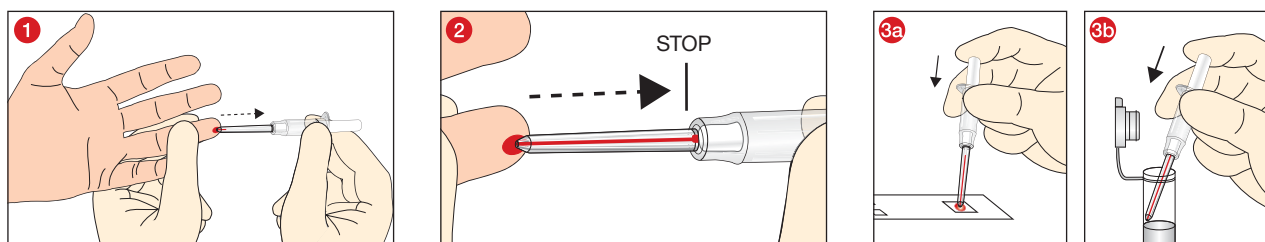
1. Use gloves, gown, goggles or other suitable protective gear to protect against bloodborne pathogens or potentially infectious materials.
2. Disinfectant for cleaning the puncture site (follow the facility's guidelines for preparing the puncture site for taking blood samples). Do not use any alcohol-based cleaning materials if the specimens are to be used for a blood alcohol test.
3. Dry, clean swabs.
4. Safety lancet, incision lancet.
5. Sharps disposal container.
6. Appropriate test kit.

Capillary technique with the Minivette® POCT

General instructions:

WEAR GLOVES TO MINIMISE THE RISK OF EXPOSURE DURING BLOOD COLLECTION.

1. Select a suitable Minivette® POCT for the required specimen.
2. Select a site for the puncture with a (safety) lancet. Please follow the guidelines of the facility for preparing the puncture site.

1. Hold the Minivette® POCT by its sides under the wings in a horizontal or slightly tilted position. The ventilation hole at the end of the plunger should not be covered when drawing up the drops of blood with the capillary tip. Do not push in the plunger and fill the capillary with no air bubbles present.
2. Blood collection stops automatically when the capillary is filled with blood up to the white filter.
- 3a. Place the capillary tip on the test field and gently press down the plunger to dispense the whole blood sample onto the test cassette.
- 3b. Alternatively, dispense the sample into a micro tube.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled capillary blood collection systems must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Product-specific standards and guidelines as amended/updated

CLSI* GP39 "Tubes and Additives for Venous and Capillary Blood Specimen Collection", Approved Standard.

CLSI* GP42 "Collection of Capillary Blood Specimens", Approved Standard.

*CLSI (Clinical and Laboratory Standards Institute)

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE marking
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Store away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product must be reported to the manufacturer and the competent national authority.

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