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DIN EN 14254: *In-vitro* diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI* GP16 "Urinalysis Approved Guideline"
 *CLSI (Clinical and Laboratory Standards Institute)

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Instructions for Use – SARSTEDT V-Monovette® Urine, urine cup and collection container with integrated transfer unit and accessories

EN

Intended use

The SARSTEDT V-Monovette® Urine specimen container system is used for the closed transfer of urine from a collection receptacle with integrated transfer unit to a V-Monovette® Urine.

In the V-Monovette® Urine specimen container system, the V-Monovette® Urine, the urine cup with integrated transfer unit and the urine collection container with integrated transfer unit are used as specimen containers for the purpose of collecting, transporting and storing urine for *in vitro* diagnostic tests. In addition, the V-Monovette® Urine is also suitable for processing the urine specimen (e.g. by centrifugation).

With very small fill volumes below the minimum fill volume, the separate transfer unit may be used as a means for transferring the specimen from a urine cup or a collection container to a V-Monovette® Urine.

The products are available individually and can be used together.

The V-Monovette® Urine specimen container system is intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The following individual components are provided for the V-Monovette® Urine specimen container system:

V-Monovette® Urine

Name	Colour code	Description
Containers without additive		
V-Monovette® Urine Z 10 ml	yellow	10 ml, 100x15 mm, round base
V-Monovette® Urine Z 9.5 ml		9.5 ml, 100x15 mm, conical base
V-Monovette® Urine Z 6 ml		6 ml, 100x13 mm, round base
V-Monovette® Urine Z 4 ml		4 ml, 75x13 mm, round base
Containers with additive		
V-Monovette® Urine boric acid 10 ml	green	10 ml, 100x15 mm, round base
V-Monovette® Urine boric acid 4 ml		4 ml, 75x13 mm, round base

SARSTEDT V-Monovette® Urine Z

The V-Monovette® Urine Z consists of a transparent plastic container with round or conical base and a colour-coded V-Monovette® membrane cap. Each V-Monovette® Urine Z is delivered with a defined vacuum corresponding to the nominal volume and a paper label. In addition, various versions of the V-Monovette® Urine Z are available for different sample volumes. They do not contain any stabilising additives. The V-Monovette® Urine Z is sterile.

SARSTEDT V-Monovette® Urine with boric acid

The V-Monovette® Urine with boric acid consists of a transparent plastic container with a round base and a colour-coded V-Monovette® membrane cap. Each V-Monovette® Urine with boric acid is delivered with a defined vacuum corresponding to the nominal volume and a paper label. In addition, various versions of the V-Monovette® Urine with boric acid are available for different sample volumes. The V-Monovette® Urine with boric acid is sterile.


The product is prepared with boric acid. The average concentration of boric acid for a filled Monovette® Urine with boric acid is 1.5% and stabilises the growth of pathogenic microorganisms in the urine at room temperature for up to 48 hours without impairing their viability.

Studies were conducted for selected strains of bacteria:

- *Escherichia coli*
- *Klebsiella pneumoniae*
- *Proteus mirabilis*
- *Streptococcus faecalis*
- *Candida albicans*

Information on product-specific studies is available on the SARSTEDT homepage: www.sarstedt.com/produkte/diagnostik/urin/urin-monovetter/

Collection receptacles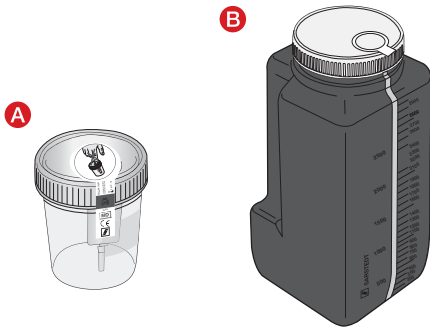

Type	Description
A	Urine cup with integrated transfer unit
B	Urine collection container with integrated transfer unit, 3 l, brown, light protection, graduation marks, inspection strip, recessed holders, enclosed self-adhesive patient label, label language depending on the order number (DE, EN, FR)

SARSTEDT Urine cup with integrated transfer unit

The urine cup with integrated transfer unit is available for a fill volume of 100 ml and consists of a transparent plastic container, graduated and with a labelling field. Integrated into the yellow screw cap is a transfer unit with a medical-grade stainless steel disposable needle providing a tight connection for the closed transfer of the urine specimen into a V-Monovette® Urine. A safety label protects the user from reaching into the transfer unit and, if intact, additionally indicates the authenticity of the urine cup. The product is available sterile and non-sterile.

SARSTEDT Urine collection container with integrated transfer unit

The urine collection container with integrated transfer unit is available for a fill volume of 3000 ml and consists of a brown-coloured (light protection) graduated plastic container with a wide opening diameter, inspection strip and recessed holders. Integrated into the yellow screw cap is a transfer unit with a medical-grade stainless steel disposable needle providing a tight connection for the closed transfer of the urine specimen into a V-Monovette® Urine. A safety label protects the user from reaching into the transfer unit.

A self-adhesive patient label is enclosed with each urine collection container with integrated transfer unit. The product is non-sterile.

Transfer unit

The transfer unit is a plastic holder containing an attachment with a needle and can be used for filling a V-Monovette® Urine. The product is non-sterile.

Accessories for the SARSTEDT V-Monovette® Urine specimen container system

Self-adhesive label for urine collection container with integrated transfer unit

Order no.	Dimensions	Version	Packaging
84.1402	150x55 mm	Label in German	300/inner carton (10x30/bag)
84.1402.001	150x55 mm	Label in English	300/inner carton (10x30/bag)
84.1402.002	150x55 mm	Label in French	300/inner carton (10x30/bag)
84.1402.003	150x55 mm	Label in Dutch	300/inner carton (10x30/bag)

Safety information and warnings

NOTE FOR STERILE VERSIONS: DO NOT USE THE PRODUCT IF THE SAFETY LABEL IS DAMAGED.

- General precautions: Use gloves and other general personal protective equipment to protect yourself from urine and possible exposure to pathogens transmitted by biological samples.
- Handle all biological specimens and sharp collection utensils (needles) according to the guidelines and procedures in your facility.
In case of direct contact with biological specimens or a needlestick injury, consult a doctor due to the risk of transmission of HIV, HCV, or HBV or other infectious diseases.
Always follow the safety guidelines and procedures of your facility.
- The products are intended for single use. Dispose of the products and all collection utensils in containers for hazardous biological waste.
- Removing the membrane cap of the V-Monovette® for manual filling of the V-Monovette® Urine impairs its sterility and is not recommended.
- The safety label on a screw cap with integrated transfer unit should remain on the transfer unit for the entire period and be removed only for the purpose of transferring urine to a V-Monovette® Urine.
To prevent needlestick injuries, never put fingers in the transfer unit.
- Always store the filled urine cup with integrated transfer unit and the collection container with integrated transfer unit in an upright position, closed and out of the reach of children.
- Do not use the products after the expiry date. The expiry date is the last day of the month and year as indicated.

Please note the safety data sheets: www.sarstedt.com/en/download/safety-data-sheets/

Storage

Store the products at room temperature.

Transport

The products can be transported together with an ADR-compliant (Packing instruction P650) secondary vessel.

Limitations

- Urine test strips belong to the most frequently used diagnostic accessories.
Before using a test strip, the manufacturer's information and possible limitations must be taken into account.
- The first morning urine is best suited for urine diagnostics. In the ideal case, there should be at least 4 hours between the previous urination and collecting the urine specimen.
- Midstream urine specimens are recommended, since they show lower contamination from urethral and/or surrounding flora compared to spontaneous urine.
Detection of bacteria (type and number) and possibly the result of a test strip analysis may be adversely influenced by e.g. genital and/or urethral contamination.
- Where possible, the urine specimens should be collected prior to beginning antibacterial therapy.
- The storage duration and temperature for urine specimens depend on the expiry date of the parameters for investigation.
In the absence of additional stabilisation, urine analysis (with the exception of 24-hour urine collection) should be carried out within one to two hours.
- The appropriate storage and stabilisation of the urine during a 24-hour collection period depend on the parameters to be examined.
The laboratory should provide precise information on this.
- Cooling of spontaneous and midstream urine specimens at 2 to 8 °C is permitted and may prolong sample stability for some analytes.
There are exceptions for light-sensitive parameters, for example bilirubin and urobilinogen. Store away from light!
- Cooling at 2 to 8 °C can promote the precipitation of amorphous urates or phosphates in the urine specimen and impair urine microscopy.
- Do not freeze urine specimens if they are to be used for urine sediment.
- For microbiological culture diagnostics, if there is no possibility of transport or immediate sample processing, storing the urine either cooled at 2 to 8 °C or in a V-Monovette® with boric acid is recommended.
- The suitability of the urine specimen for the analysis of trace elements/metals has not been tested.
- The volume that is transferred to a V-Monovette® Urine can vary depending on different physical factors such as altitude, ambient temperature, the remaining shelf life of the product and the filling procedure.
- The V-Monovette® Urine with boric acid must be filled up to the filling line to ensure the necessary ratio of urine to preservative.
A deviation of +/-10% from the filling line is permissible. Underfilling can lead to false results.

Specimen collection and handling

READ ALL OF THIS DOCUMENT BEFORE YOU START URINE COLLECTION AND / OR URINE TRANSFER.

Equipment required for urine collection

A: Urine cup with transfer unit


- Urine cup with transfer unit
- Clean cloths or flannels and towel

B: Urine collection container with transfer unit

If needed, collection cup for urine portions, for example 500 ml cup, Art. no. 75.1356.

A: Procedure for collecting midstream urine using a urine cup with transfer unit

- Wash your hands thoroughly and then the genital area. Dry yourself off with a clean towel.
- Ensure that the safety label is intact.
- Then open the lid of the urine cup by turning it anticlockwise. Place the lid in a hygienic place.
Ensure that the inside of the lid is not contaminated.
- First pass a small quantity of urine into the toilet (Fig. 4a), then catch the midstream urine in the urine cup without interrupting the urine flow (Fig. 4b).
Then fully empty your bladder into the toilet (Fig. 4c).
- Close the cap of the urine cup tightly by turning it clockwise. Be careful not to contaminate the inside of the lid.
Label the urine cup with your name and immediately deliver the tightly closed urine cup with the specimen to the responsible person, or proceed as instructed by your doctor.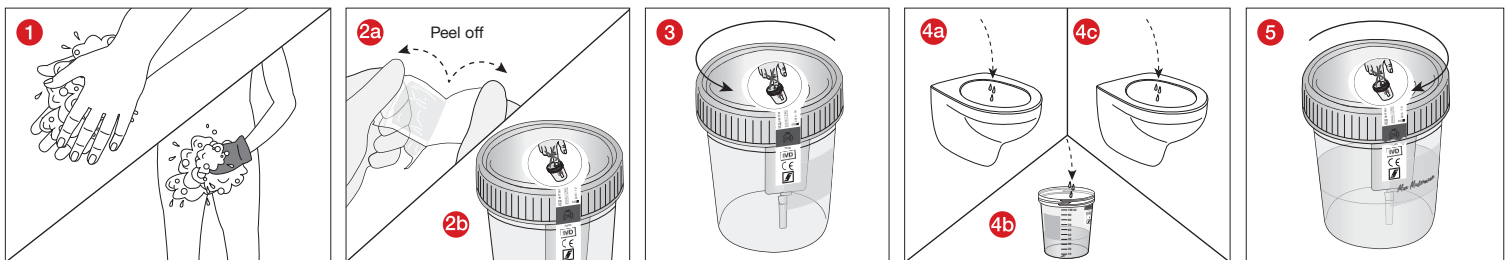

B: Procedure for a 24-hour urine collection using a urine collection container with transfer unit

Empty your bladder in the toilet in the morning when you get up. Note the date and time. Use the enclosed patient label for this and complete the patient data.

1. Collect the next urine in the collection container. You may use a small collection cup, but one that is sufficiently large and transfer all the urine to the collection container.
2. Close the collection container well.
3. Collect all further urine, even when having a bowel movement, and put it in the collection container.
4. Close the collection container well every time and swirl it back and forth. Make sure to store it as recommended by your doctor.
5. Go to the toilet the next day at the time noted on the day before, collect this urine and add it to the collection container. The collection phase is then complete.
6. Bring the filled collection container to your doctor or proceed as instructed by your doctor.

Procedure for transferring the urine to a V-Monovette® Urine**Material required for the urine transfer**

1. Gloves, gown, goggles or other suitable protective gear to protect against potentially infectious materials.
2. Required quantity of V-Monovettes® Urine.
3. Transfer unit (Art. no. 11.1240); only necessary if the minimum fill volume has not been reached:
Urine cup with transfer unit < 20 ml
Urine collection container with transfer unit < 700 ml
4. Urine cup (for example, Art. no. 75.562.005) only necessary if the following fill volume has not been reached:
Urine collection container with transfer unit < 700 ml
5. Sharps disposal container for disposal of used urine transfer unit.
6. Disposal container for hazardous biological waste.




Recommended order of collection

If multiple V-Monovettes Urine are to be filled from a primary container, the following order is recommended:

V-Monovette® Urine	Intended use
1. V-Monovette® Urine Z (without stabiliser)	Microbiological tests
2. V-Monovette® Urine with boric acid (with stabiliser)	
3. V-Monovette® Urine Z (without stabiliser)	Chemical tests, strip tests and urine particle analysis


A1: From a urine cup with transfer unit

1. Prepare the urine specimen for removal by gently swirling the urine cup. Make sure that the fill volume of the closed urine cup is 20 ml or more.
If not, proceed as described under A2.
2. Remove the safety label from the top of the urine cup to allow access to the integrated transfer unit (Fig. 1). Stick the safety label back on the opening after removing the urine to reseal it.
NOTE: The minimum filling quantity required for filling a V-Monovette® Urine is 20 ml for the closed urine cup.
3. Place the urine cup upright on a solid surface. Guide the V-Monovette® Urine with the cap first into the integrated transfer unit and make sure that the needle penetrates the stopper of the V-Monovette® Urine (Fig. 2). The V-Monovette® Urine starts filling automatically according to the defined vacuum. If no urine flows or the flow stops before a sufficient fill level is achieved, the following steps are recommended:
 - a. Press the V-Monovette® Urine forward firmly again to make sure that the stopper is completely penetrated.
 - b. Secure the V-Monovette® Urine in the transfer unit during the filling process by pressing against the bottom of the V-Monovette® Urine.
 - c. If the urine still does not flow, remove the V-Monovette® Urine and use a new V-Monovette® Urine.
4. Hold the V-Monovette® Urine in place until no more urine flows into it and only then remove it from the integrated transfer unit.
5. Only for V-Monovette® Urine with boric acid: Swirl the V-Monovette® Urine at least five times or until the specimen is completely mixed with the preservative (Fig. 3).
6. Dispose of the urine cup in a container for hazardous biological waste.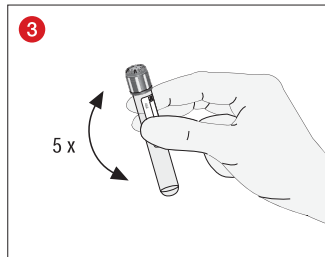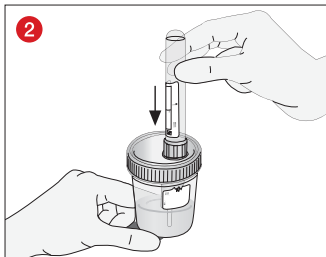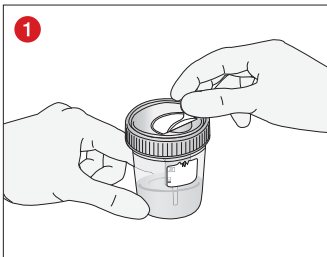


A2: When using a transfer unit (Art. no. 11.1240)

1. Prepare the urine specimen for removal by gently swirling the urine cup.
2. Open the urine cup. Dip the tip of the urine transfer unit into the urine specimen (Fig. 1).
3. Guide the V-Monovette® Urine with the cap first into the transfer unit and make sure that the needle penetrates the stopper of the V-Monovette® Urine (Fig. 3).
The V-Monovette® Urine starts filling automatically according to the defined vacuum. If no urine flows or the flow stops before a sufficient fill level is achieved, the following steps are recommended:
 - a. Press the V-Monovette® Urine forward firmly again to make sure that the stopper is completely penetrated.
 - b. Secure the V-Monovette® Urine in the transfer unit during the filling process by pressing your thumb against the bottom of the V-Monovette® Urine.
 - c. If the urine still does not flow, remove the V-Monovette® Urine and use a new V-Monovette® Urine.
4. Hold the V-Monovette® Urine in place until no more urine flows into it.
5. Remove the V-Monovette® Urine from the transfer unit and only then take the transfer unit out of the fluid.
6. Only for V-Monovette® Urine with boric acid: Swirl the V-Monovette® Urine at least five times or until the specimen is completely mixed with the preservative (Fig. 3).
7. Dispose of the transfer unit in a sharps disposal container (Fig. 4).
8. Close the urine cup and dispose of it in a container for hazardous biological waste.

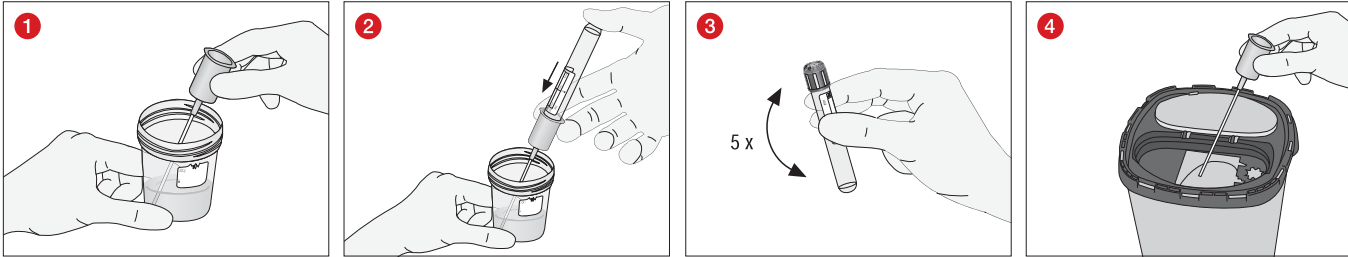**B1: From a urine collection container with transfer unit, fill volume > 700 ml**


1. Prepare the urine specimen for removal by gently swirling the urine collection container.
2. Remove the safety label from the top of the collection container to allow access to the integrated transfer unit. Stick the safety label back on the opening after removing the urine to reseal it.
NOTE: The minimum filling volume required to fill a V-Monovette® Urine depends on the position of the collection container (Fig. 1).
3. Select a position of the collection container appropriate for the filling volume (Fig. 2 – 4) and only then guide the V-Monovette® Urine with the cap first into the integrated transfer unit. Make sure that the needle penetrates the stopper of the V-Monovette® Urine. The V-Monovette® Urine starts filling automatically according to the defined vacuum.
If no urine flows or the flow stops before a sufficient fill level is achieved, the following steps are recommended:
 - a. Press the V-Monovette® Urine forward firmly again to make sure that the stopper is completely penetrated.
 - b. Secure the V-Monovette® Urine in the transfer unit during the filling process by pressing against the bottom of the V-Monovette® Urine.
 - c. If the urine still does not flow, remove the V-Monovette® Urine and use a new V-Monovette® Urine.
4. Hold the V-Monovette® Urine in place until no more urine flows into it and only then remove it from the integrated transfer unit.
5. Dispose of the urine collection container in a suitable container for hazardous biological waste.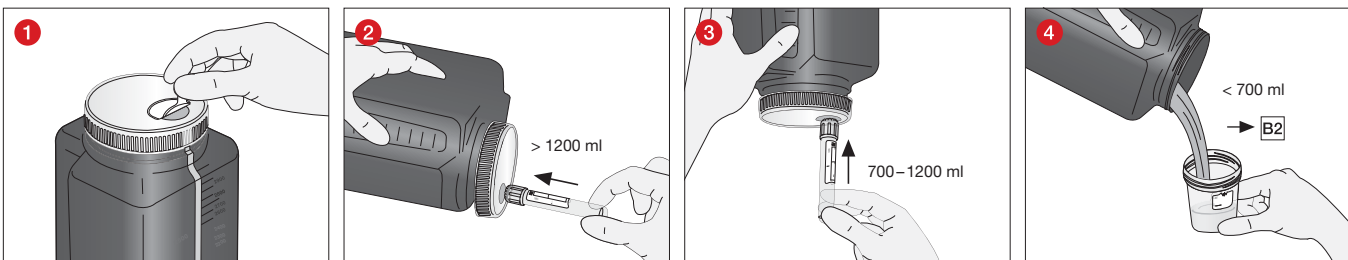

B2: From a urine collection container with transfer unit, fill volume <700 ml

1. Prepare the urine specimen for removal by gently swirling the urine collection container and transfer a sufficient quantity to a urine cup.
2. Dip the tip of the urine transfer unit into the urine specimen (Fig. 1).
3. Guide the V-Monovette® Urine with the cap first into the transfer unit and make sure that the needle penetrates the stopper of the V-Monovette® Urine (Fig. 2).
The V-Monovette® Urine starts filling automatically according to the defined vacuum.
If no urine flows or the flow stops before a sufficient fill level is achieved, the following steps are recommended:
 - a. Press the V-Monovette® Urine forward firmly again to make sure that the stopper is completely penetrated.
 - b. Secure the V-Monovette® Urine in the transfer unit during the filling process by pressing your thumb against the bottom of the V-Monovette® Urine.
 - c. If the urine still does not flow, remove the V-Monovette® Urine and use a new V-Monovette® Urine.
4. Hold the V-Monovette® Urine in place until no more urine flows into it.
5. Remove the V-Monovette® Urine from the transfer unit and only then take the transfer unit out of the fluid.
6. Dispose of the transfer unit in a container for hazardous biological waste (Fig. 3).
7. Close the urine cup and the urine collection container with transfer unit and dispose of both in a container for hazardous biological waste.

NOTE: It is also possible to use the urine cup with integrated transfer unit. For this, proceed as described under A1.

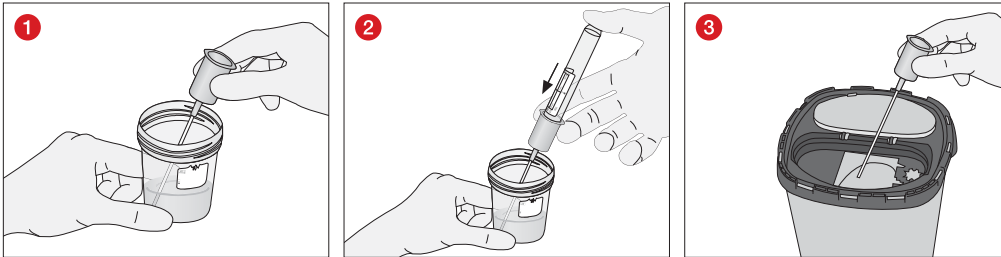Centrifugation

WARNING! SARSTEDT V-Monovettes Urine are designed for maximum 4000 x g (RCF - relative centrifugal force – g-force).

Centrifuge inserts are to be selected according to the size of the V-Monovettes Urine used. The relative centrifugal force has the following ratio to the selected rpm:

$$\text{RCF (g-force)} = 11.2 \times r \times (\text{rpm}/1000)^2$$

"RCF": "Relative centrifugal force"

"rpm": "Revolutions per minute" (min⁻¹)

"r" [in cm]: "Radius of rotation from the middle of the centrifuge to the bottom" of the V-Monovette® Urine.

Use only suitable tube holders or inserts. Centrifuging V-Monovettes Urine with cracks or centrifuging at an excessive centrifugal force can lead to breakage of the V-Monovettes Urine, which can release potentially hazardous substances.

V-Monovettes Urine should be centrifuged in accordance with the centrifugation conditions listed below. If other conditions are used, they must be validated by the user themselves. It must be ensured that the V-Monovettes Urine fit well in the centrifuge inserts. V-Monovettes Urine that protrude beyond the insert can catch on the centrifuge head and be broken. Ensure that the centrifuge is filled evenly. Please refer to the centrifuge instructions for use for further information.

CAUTION! Do not remove broken V-Monovettes Urine by hand.

Refer to the centrifuge instructions for use for information on disinfecting the centrifuge.

Centrifugation recommendation:

The recommended RCF force (g-force) for urine specimens for sediment extraction is 400 x g for 5 minutes.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled urine cups must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).
5. Please take note of the information in the safety data sheet.

Product-specific standards and guidelines as amended / updated

DIN EN 14254: *In vitro* diagnostic medical devices – Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI* GP16 “Urinalysis Approved Guideline”
 *CLSI (Clinical and Laboratory Standards Institute)

Further literature:

European Urinalysis Guidelines: Scand J. Clin. Lab. Invest 2000; 60: 1–96.

Key for symbols and labels:

-  Article number
-  Batch number
-  Use by
-  CE marking
-  *In vitro* diagnostic device
-  Follow the instructions for use
-  If reused: Risk of contamination
-  Store away from sunlight
-  Store in a dry place
-  Manufacturer
-  Country of manufacture

The following also applies to sterile products:

-  Single sterile barrier system with external protective packaging
-  Sterile fluid path
-  Do not re-sterilise

Technical modifications reserved.

All serious incidents relating to the product must be reported to the manufacturer and the competent national authority.

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