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SARSTEDT Microvette® CB 200 BSG

| | | |
|-----------|--|---------|
| DE | Gebrauchshinweis – SARSTEDT Microvette® CB 200 BSG | 2 - 4 |
| EN | Instructions for use – SARSTEDT Microvette® CB 200 ESR | 5 - 7 |
| BG | Инструкции за употреба – SARSTEDT Microvette® CB 200 BSG | 8 - 10 |
| CS | Návod k obsluze – SARSTEDT Microvette® CB 200 BSG | 11 - 13 |
| DA | Brugsanvisning – SARSTEDT Microvette® CB 200 BSG | 14 - 16 |
| EL | Οδηγίες χρήσεως – SARSTEDT Microvette® CB 200 BSG | 17 - 19 |
| ES | Instrucciones de uso – SARSTEDT Microvette® CB 200 VSG | 20 - 22 |
| ET | Kasutusjuhend – verevõtusüsteem SARSTEDT Microvette® CB 200 BSG | 23 - 25 |
| FR | Mode d'emploi – Microvette® CB 200 BSG SARSTEDT | 26 - 28 |
| HR | Uputa za upotrebu – SARSTEDT Microvette® CB 200 BSG | 29 - 31 |
| HU | Használati utasítás – SARSTEDT Microvette® CB 200 BSG | 32 - 34 |
| IT | Istruzioni d'uso – Microvette® CB 200 BSG SARSTEDT | 35 - 37 |
| KO | 사용 설명서 – SARSTEDT Microvette® CB 200 ESR | 38 - 40 |
| LT | Naudojimo instrukcijos – SARSTEDT „Microvette® CB 200 BSG“ | 41 - 43 |
| LV | Lietošanas norādes – SARSTEDT Microvette® CB 200 BSG | 44 - 46 |
| NL | Gebruiksaanwijzing – SARSTEDT Microvette® CB 200 BSG | 47 - 49 |
| NO | Bruksanvisning – SARSTEDT Microvette® CB 200 BSG | 50 - 52 |
| PL | Instrukcja obsługi – próbówka Microvette® CB 200 BSG SARSTEDT | 53 - 55 |
| PT | Instruções de utilização – SARSTEDT Microvette® CB 200 BSG | 56 - 58 |
| RO | Instrucțiuni de utilizare – Microvette® CB 200 BSG SARSTEDT | 59 - 61 |
| RU | Инструкция по применению – SARSTEDT Microvette® CB 200 COЭ | 62 - 64 |
| SK | Návod na Použitie – SARSTEDT Microvette® CB 200 ESR | 65 - 67 |
| SL | Navodila za uporabo – Epruveta Microvette® CB 200 BSG SARSTEDT | 68 - 70 |
| SV | Bruksanvisning – SARSTEDT Microvette® CB 200 SR | 71 - 73 |
| TH | คำแนะนำในการใช้งาน – ระบบการเจาะเลือดจากเส้นเลือดฝอย Microvette® CB 200 BSG ของ SARSTEDT | 74 - 76 |
| TR | Kullanım için talimatlar – SARSTEDT Microvette® CB 200 BSG | 77 - 79 |
| ZH | 使用说明 – SARSTEDT Microvette® CB 200 BSG | 80 - 82 |

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Intended use

The Microvette® CB 200 ESR is a blood collection system and is used with lancets to collect capillary blood using an End-to-End capillary. The Microvette® CB 200 ESR is used for the collection, manual in-house transport and processing of the capillary blood sample for measuring the blood sedimentation together with an enclosed sedimentation capillary and the corresponding ESR rack. The product is intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The Microvette® CB 200 ESR consists of a plastic container, a 200 µl capillary with integrated holder and a colour-coded plastic push-in plug in violet or black, and contains citrate as an anticoagulant (mixing ratio 1:5).

Safety information and warnings

1. General precautions: Use gloves and other general personal protective equipment to protect yourself against blood and possible exposure to pathogens transmitted by blood.
2. Handle all biological specimens and sharp blood collection utensils according to the guidelines and procedures in your facility. In case of direct contact with biological specimens or a needlestick injury, consult a doctor due to the risk of transmission of HIV, HCV, or HBV or other infectious diseases. Always follow the safety guidelines and procedures of your facility.
3. Dispose of all sharp objects for capillary blood collection in appropriate disposal containers.
4. Under- or overfilling the Microvettes leads to an incorrect ratio of blood to the preparation/additive and can result in incorrect analysis results.
5. The blood collected and processed with the Microvette® is not intended to be re-injected into the human body.
6. Do not use the product after the expiry date. The expiry date of the product is the last day of the month and year as indicated.

Storage

Store the products at room temperature.

Limitations

1. If blood samples are stored in the Microvette®, the stability of the analytes should be evaluated by the respective laboratory or taken from the instructions for use of the analysis device manufacturer.
2. For therapeutic medication, the suitability of the sample material must be checked in the manufacturer's instructions for use of the assay/analysis device.
3. The Microvette® CB 200 ESR is designed exclusively for use in the manual SARSTEDT ESR rack (Art. no. 90.1091).

Collecting and handling specimens

READ THIS DOCUMENT COMPLETELY BEFORE STARTING TO DRAW CAPILLARY BLOOD.

Preparation for drawing capillary blood and material needed:

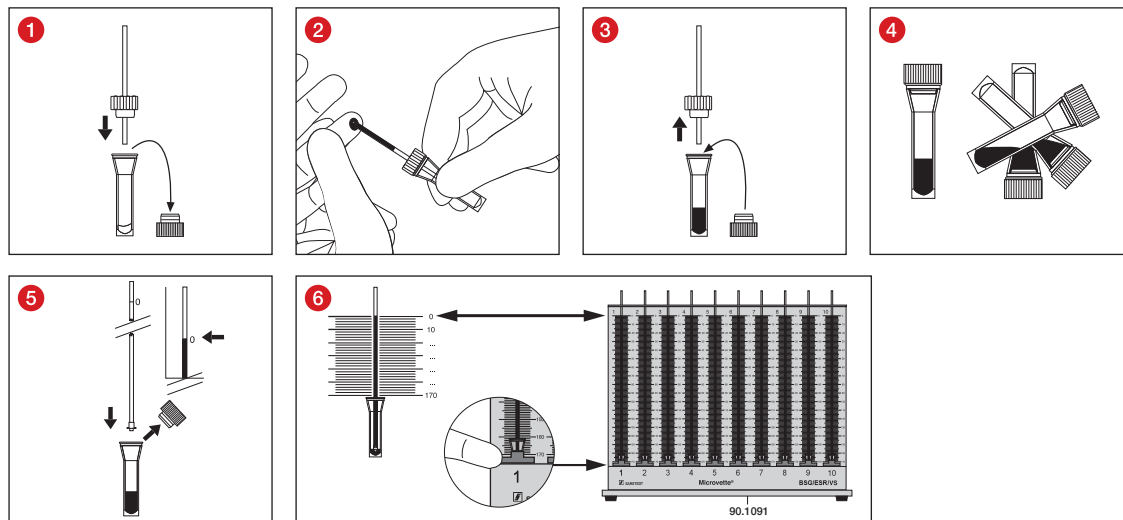
1. All required Microvettes.
2. Gloves, gown, goggles or other suitable protective wear to protect against bloodborne pathogens or potentially infectious materials.
3. Labels for specimen identification.
4. (Safety) lancet or incision lancet.
5. Disinfectant for cleaning the puncture site (follow the facility's guidelines for preparing the puncture site for taking blood samples). Do not use any alcohol-based cleaning materials if the specimens are to be used for a blood alcohol test.
6. Dry, clean swabs.
7. Plasters.
8. Sharps disposal container for the safe disposal of used material.

Capillary technique with the End-to-End capillary

General instructions:

WEAR GLOVES TO MINIMISE THE RISK OF EXPOSURE DURING CAPILLARY BLOOD COLLECTION.

1. Select a suitable Microvette® for the required specimen.
2. Select a site for the puncture with a (safety) lancet or incision lancet. Please follow the guidelines of the facility for preparing the puncture site.

1. Open the Microvette® CB 200 ESR by pulling the push-in plug upwards. Install the capillary by pressing in the holder.
2. Hold the Microvette® CB 200 ESR horizontally or tilted slightly and collect the drops of blood with the 200 µl End-to-End capillary. Stop collecting blood when the End-to-End capillary is completely filled with blood.
3. Hold the Microvette® CB 200 ESR upright so the blood flows into the collection container. Twist slightly to remove the cap, incl. the End-to-End capillary, and discard them. Close the container using the push-in plug.
4. Invert the specimen carefully.
5. Remove the push-in plug to insert the sedimentation capillary into the Microvette® CB 200 ESR. Carefully push down the sedimentation capillary. The sedimentation capillary will then fill with blood. Stop when the fill mark on the sedimentation capillary has reached exactly 0.
6. To perform the measurement, place the Microvette® CB 200 ESR on the SARSTEDT ESR rack (Art. no.: 90.1091).

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled capillary blood collection systems must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Product-specific standards and guidelines as amended/updated

CLSI* GP39 "Tubes and Additives for Venous and Capillary Blood Specimen Collection", Approved Standard.

CLSI* GP42 "Collection of Capillary Blood Specimens", Approved Standard.

CLSI* GP44 "Procedures for the Handling and Processing of Blood Specimens for Common Laboratory Tests", Approved Guideline.

WHO/DIL/LAB/99.1 Rev.02 "WORLD HEALTH ORGANIZATION, et al. Use of anticoagulants in diagnostic laboratory investigations. Geneva: World Health Organization, 2002".

*CLSI (Clinical and Laboratory Standards Institute)

Key for symbols and labels:

Article number

Batch number

Use by

CE marking

In vitro diagnostic device

Follow the instructions for use

If reused: Risk of contamination

Store away from sunlight

Store in a dry place

Manufacturer

Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product must be reported to the manufacturer and the competent national authority.

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CLSI* GP39 "Tubes and Additives for Venous and Capillary Blood Specimen Collection", Approved Standard.
 CLSI* GP42 "Collection of Capillary Blood Specimens", Approved Standard.
 CLSI* GP44 "Procedures for the Handling and Processing of Blood Specimens for Common Laboratory Tests", Approved Guideline.
 WHO/DIL/LAB/99.1 Rev02 "WORLD HEALTH ORGANIZATION, et al. Use of anticoagulants in diagnostic laboratory investigations. Geneva: World Health Organization, 2002".
 *CLSI (Clinical and Laboratory Standards Institute)

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CLSI* GP39 „Tubes and Additives for Venous and Capillary Blood Specimen Collection”, Approved Standard.

CLSI* GP42 „Collection of Capillary Blood Specimens”, Approved Standard.

CLSI* GP44 „Procedures for the Handling and Processing of Blood Specimens for Common Laboratory Tests”, Approved Guideline.

WHO/DIL/LAB/99.1 Rev02 „WORLD HEALTH ORGANIZATION, et al. Use of anticoagulants in diagnostic laboratory investigations. Geneva: World Health Organization, 2002”.

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