S-Monovette® RNA Exact

DE	Gebrauchshinweis – S-Monovette® RNA Exact	3 - 6
EN	Instructions for use – S-Monovette® RNA Exact	7 - 10
BG	Инструкции за употреба – S-Monovette® RNA Exact	11 - 14
BR	Instruções de utilização – S-Monovette® RNA Exact	15 - 18
CS	Návod k obsluze – S-Monovette® RNA Exact	19 - 22
DA	Brugsanvisning – S-Monovette® RNA Exact	23 - 26
EL	Οδηγός χρήσεως – S-Monovette® RNA Exact	27 - 30
ES	Instrucciones de uso – S-Monovette® RNA Exact	31 - 34
ET	Kasutusjuhend – S-Monovette® RNA Exact	35 - 38
FR	Mode d'emploi – S-Monovette® RNA Exact	39 - 42
HR	Uputa za upotrebu – S-Monovette® RNA Exact	43 - 46
HU	Használati utasítás – S-Monovette® RNA Exact	47 - 50
IT	Istruzioni d'uso – S-Monovette® RNA Exact	51 - 54
KO	사용 설명서 – S-Monovette® RNA Exact	55 - 58
LT	Naudojimo instrukcijos – S-Monovette® RNA Exact	59 - 62
LV	Lietošanas norādes – S-Monovette® RNA Exact	63 - 66
NL	Gebruiksaanwijzing – S-Monovette® RNA Exact	67 - 70
NO	Bruksanvisning – S-Monovette® RNA Exact	71 - 74
PL	Instrukcja obsługi – S-Monovette® RNA Exact	75 - 78
PT	Instruções de utilização – S-Monovette® RNA Exact	79 - 82
RO	Instrucțiuni de utilizare – S-Monovette® RNA Exact	83 - 86
RU	Инструкция по применению – S-Monovette® RNA Exact	87 - 90
SK	Návod na Použitie – S-Monovette® RNA Exact	91 - 94
SL	Navodila za uporabo – S-Monovette® RNA Exact	95 - 98
SV	Bruksanvisning – S-Monovette® RNA Exact	99 - 102
TH	คำแนะนำในการใช้งาน – S-Monovette® RNA Exact	103 - 106
TR	Kullanım için talimatlar – S-Monovette® RNA Exact	107 - 110
US	Instructions for use – S-Monovette® RNA Exact	111 - 114
ZH	使用说明 – S-Monovette® RNA Exact	115 - 118

S-Monovette® RNA Exact

DE

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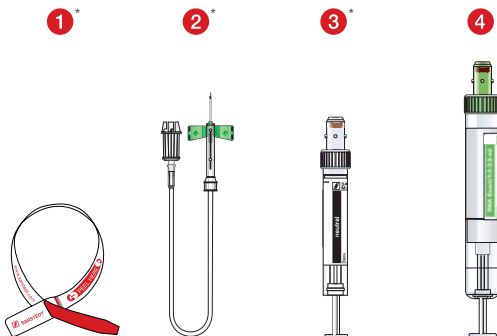
Instructions for use

Intended purpose

The S-Monovette® RNA Exact is used for venous blood collection and allows a freely selectable sample volume of up to 9.7 ml (including up to 2.4 ml of blood) to be collected. The product serves to immediately stabilize all of the RNA contained in the blood (prevents degradation and new synthesis of RNA). The S-Monovette® RNA Exact standardizes preanalytics in a closed system during storage and transport of the sample.

Required materials

To obtain venous blood for RNA analysis, you need the following products, some of which are not included with the S-Monovette® RNA Exact*:


- 1 A tourniquet, e.g. REF 95.1006
- 2 One Safety-Multifly®-Needle (REF 85.1637.235, 85.1638.235, 85.1640.235 or 85.1642.235), length: 200 mm
- 3 One empty/discard tube (e.g. S-Monovette® Neutral or other S-Monovette®) for removing air from the Safety-Multifly®-Needle tubing
- 4 S-Monovette® RNA Exact, REF 01.2048.001

Handling


The stabilizer solution that comes in the S-Monovette® RNA Exact is harmful to health. Please follow the procedure described here and use only the prescribed materials for blood collection. Disinfect beforehand according to your in-house hygiene plan. The S-Monovette® RNA Exact should always be filled as the last S-Monovette® for multiple withdrawals. Blood can be collected with the S-Monovette® RNA Exact using the aspiration technique or the vacuum technique.

When using the aspiration technique, fill the S-Monovette® RNA Exact with the desired amount of blood after connection by slowly retracting the plunger.

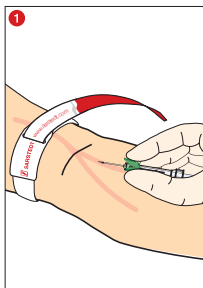
When using the vacuum technique, retract the plunger to the rear latching position („click“) before connecting the S-Monovette® RNA Exact and break it off.

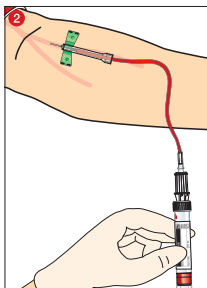
1) Venipuncture

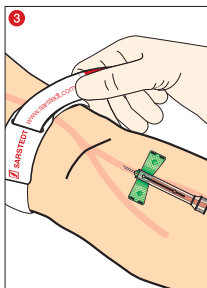
2) Fill Safety-Multify®-Needle and empty discard tubes with blood.

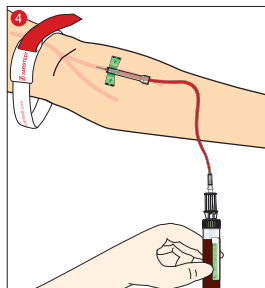
3) Now loosen the tourniquet and connect S-Monovette® RNA Exact.

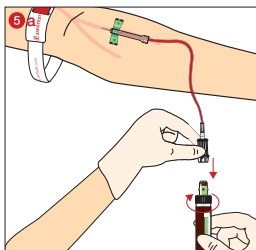
4) Aspiration technique: Connect the S-Monovette® RNA Exact and fill by slowly retracting the plunger. Vacuum technique: Connect the evacuated S-Monovette® RNA Exact and hold it upright below arm level until it is filled.

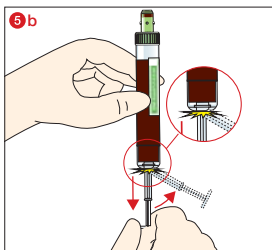5a) Disconnecting. At maximum filling of the S-Monovette® RNA Exact, the fill level is in the region of the upper edge of the label.

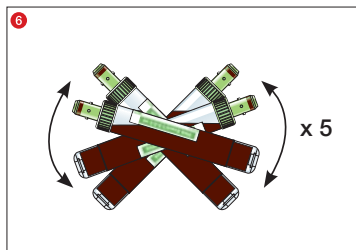5b) In the aspiration technique, (bring the plunger, if necessary, into the end position and) break the plunger off.

6) Invert filled S-Monovette® RNA Exact at least 5 times so that the preparation mixes well with the blood sample.

Sample shipment

Since sampling is often decentralized, it is frequently necessary to transport samples to the laboratory after blood collection. For safe transport, SARSTEDT offers a complete packaging system for the S-Monovette® that has been tested and meets the requirements of packaging regulation P 650 of the ADR, RID, ICAO and IATA:

- Protective vessel with suction insert (REF 78.898) and closure (REF 65.679) as secondary vessel
- Shipping box (REF 95.900, 95.901 & 95.902)
- Shipping packaging for refrigerated transport (REF 95.995, 95.1123, 95.1124 or 95.1625)
- Styrofoam cover for refrigerated transport (REF 95.1011)

Sample storage

In the S-Monovette® RNA Exact, the RNA profile of the blood sample remains stable for at least:

- 5 days at room temperature (18 °C to 22 °C)
 - 14 days at 2 °C to 8 °C
 - long-term storage at -40 °C to -80 °C
- The S-Monovette® RNA Exact consists of temperature-stable polypropylene and is in principle well suited for freezing.
 - Nevertheless, care should be taken to ensure that the samples are frozen upright in an open rack. Insulating polystyrene foam racks or closed block racks must not be used.
 - Samples must be frozen directly at -40 °C or lower (e.g. -80 °C).
 - Do not heat samples above room temperature to thaw. After defrosting, invert to gently mix the sample.

Suitable products for RNA isolation are:

1. Manual isolation systems

- NucleoSpin® RNA Blood Midi Kit, Macherey-Nagel, REF 740210.20

2. Automated isolation systems

- chemagic Total RNA 9k Kit H24, Perkin Elmer, REF CMG-1084-S
- InviMag Blood RNA Kit /IG (8x12), Invitex-Molecular, REF 2463120100

Further information can be found on the SARSTEDT website and in the white paper *Impact of RNA Stabilizing Blood Collection Tubes on Gene Expression Data Validity – A Comparison of S-Monovette® RNA Exact, PAXgene™ Blood RNA Tubes & Tempus™ Blood RNA Tubes*.

Warnings

- Never loosen the tourniquet while the S-Monovette® RNA Exact is connected.
- Blood and blood components collected and processed in the S-Monovette® RNA Exact are not suitable for infusion or introduction into the human body.
- The usual precautions for handling laboratory reagents must be observed.
- Use S-Monovette® only for blood collection and not for injection.

Danger

Hazard statements

H314	Causes severe skin burns and eye damage.
H332	Harmful if inhaled.
H412	Harmful to aquatic life with long lasting effects.

Precautionary statements

P260	Do not breathe dust/fume/gas/mist/vapours/spray.
P264	Wash hands thoroughly after handling.
P273	Avoid release to the environment.
P280	Wear protective gloves/protective clothing/eye protection/face protection.
P301+P330+P331	IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.
P303+P361+P353	IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower.
P363	Wash contaminated clothing before reuse.
P304+P340	IF INHALED: Remove person to fresh air and keep comfortable for breathing.
P310	Immediately call a POISON CENTER/doctor.
P305+P351+P338	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P310	Immediately call a POISON CENTER/doctor.
P405	Store locked up.
P501	Dispose of contents/container to waste disposal site according to legal regulations.

Special labelling of certain mixtures

EUH032	Contact with acids liberates very toxic gas.
--------	--

Safety data sheet at www.sarstedt.com

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority where the user is located.

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Danger

Hazard statements

Causes severe skin burns and eye damage
Harmful if inhaled

Precautionary statements

Do not breathe dust/fume/gas/mist/vapors/spray.
Wash thoroughly after handling.
Wear protective gloves/protective clothing/eye protection/face protection.
If swallowed: Rinse mouth. Do NOT induce vomiting.
If on skin (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.
Wash contaminated clothing before reuse.
If inhaled: Remove person to fresh air and keep comfortable for breathing. Immediately call a poison center/doctor.
If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a poison center/doctor.
Store locked up.
Dispose of contents/container to waste disposal site according to legal regulations.

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SARSTEDT AG & Co. KG