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SARSTEDT Salivette® Cortisol

DE	Gebrauchshinweis – SARSTEDT Salivette® Cortisol	2 - 4
EN	Instructions for use – SARSTEDT Salivette® Cortisol	5 - 7
BG	Инструкции за употреба – SARSTEDT Salivette® Cortisol	8 - 10
CS	Návod k obsluze – SARSTEDT Salivette® Cortisol	11 - 13
DA	Brugsanvisning – SARSTEDT Salivette® Cortisol	14 - 16
EL	Οδηγίες χρήσεως – SARSTEDT Salivette® Cortisol	17 - 19
ES	Instrucciones de uso – SARSTEDT Salivette® Cortisol	20 - 22
ET	Kasutusjuhend – SARSTEDT Salivette® Cortisol	23 - 25
FR	Mode d'emploi – SARSTEDT Salivette® Cortisol	26 - 28
HR	Uputa za upotrebu – SARSTEDT Salivette® Cortisol	29 - 31
HU	Használati utasítás – SARSTEDT Salivette® Cortisol	32 - 34
IT	Istruzioni d'uso – SARSTEDT Salivette® Cortisol	35 - 37
KO	사용 설명서 – SARSTEDT Salivette® Cortisol	38 - 40
LT	Naudojimo instrukcijos – SARSTEDT Salivette® Cortisol	41 - 43
LV	Lietošanas norādes – SARSTEDT Salivette® Cortisol	44 - 46
NL	Gebruiksaanwijzing – SARSTEDT Salivette® Cortisol	47 - 49
NO	Bruksanvisning – SARSTEDT Salivette® Cortisol	50 - 52
PL	Instrukcja obsługi – SARSTEDT Salivette® Cortisol	53 - 55
PT	Instruções de utilização – SARSTEDT Salivette® Cortisol	56 - 58
RO	Instrucțiuni de utilizare – SARSTEDT Salivette® Cortisol	59 - 61
RU	Инструкция по применению – SARSTEDT Salivette® Cortisol	62 - 64
SK	Návod na použitie – SARSTEDT Salivette® Cortisol	65 - 67
SL	Navodila za uporabo – SARSTEDT Salivette® Cortisol	68 - 70
SV	Bruksanvisning – SARSTEDT Salivette® Cortisol	71 - 73
TH	คำแนะนำในการใช้งาน – SARSTEDT Salivette® Cortisol	74 - 76
TR	Kullanım için talimatlar – SARSTEDT Salivette® Cortisol	77 - 79
ZH	使用说明 – SARSTEDT Salivette® Cortisol	80 - 82

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Literatur

SARSTEDT Whitepaper <https://www.sarstedt.com/en/products/diagnostic/salivasputum/Literature/> „Comparing cortisol levels of saliva samples collected with the passive drool collection method with the Salivette® Cortisol“

Kirschbaum, Hellhammer "Salivary Cortisol", Encyclopedia of STRESS 1st ed, vol 3:379-383, 2000 (cited in 2nd ed. 405-409 2007; doi.org/10.1016/B978-012373947-6.00334-2)

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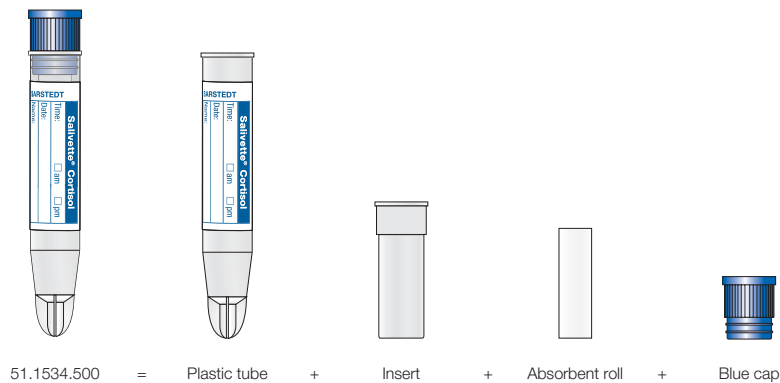
Intended use

The Salivette® Cortisol is a specimen tube for saliva samples and is used for the collection, transport, processing and storage of saliva for *in vitro* diagnostic determination of cortisol. The product is intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The Salivette® Cortisol consists of a transparent conical bottom plastic tube, an insert, an absorbent roll and a blue cap.

Safety information and warnings

1. **The product must not be used in children under 3 years of age or in patients at increased risk of choking.**
2. General precautions: Use gloves and other general personal protective equipment to protect yourself against possible exposure to potentially infectious sample material. Always follow the safety guidelines and procedures of your facility.
3. The saliva collected and processed with the Salivette® Cortisol is not intended to be returned to the human body.
4. Do not use Salivette® Cortisol after the expiry date. The expiry date of Salivette® Cortisol is the last day of the month and year as indicated.

Storage

Store Salivette® Cortisol at room temperature before use.

Transport

The product can be transported manually in-house or with an ADR-compliant (packaging instruction P650) secondary container. The ADR-compliant (packaging instruction P650) shipping container, Art. no. 78.898, with absorbent insert plus screw cap, Art. no. 65.679, and shipping box/bag is recommended for Salivette® Cortisol specimen transport.

For the patient:

To return the saliva sample in the tightly closed Salivette® Cortisol, follow the doctor's instructions and use the secondary container provided by the doctor and the shipping box/bag. If it is not returned immediately, store this Salivette® Cortisol in the refrigerator.

Limitations

The saliva sample should be taken at least 60 minutes after a meal (liquid/solid food intake), brushing teeth or taking oral medication, in order to avoid the saliva being contaminated with interfering substances.

The storage duration and temperature depend on the shelf life for the parameters to be tested. Therefore, and since the onset of bacterial growth in saliva can be expected after just a few hours at room temperature, it is recommended that the Salivette® Cortisol be sealed tightly after sampling and either centrifuged and analysed within 4 hours or placed immediately in a refrigerator until further processing.

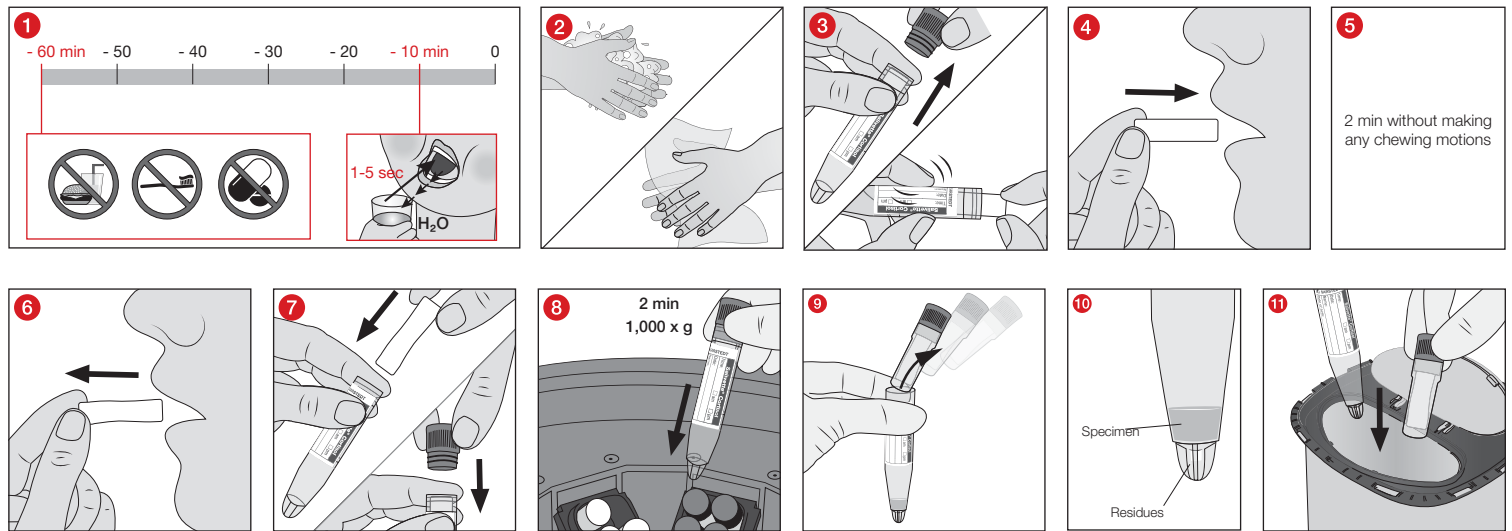
Specimen collection and handling

READ ALL OF THIS DOCUMENT BEFORE STARTING TO COLLECT SALIVA.

Material required to collect a specimen:

1. Gloves, gown, goggles or other suitable protective gear to protect against pathogens transmitted via saliva or potentially infectious materials.
2. Number of Salivette® Cortisol tubes as requested by the doctor.
3. Fill out information on sample identification (on label).

Sample collection

- 1 The saliva sample should be taken at least 60 minutes after a meal (liquid/solid food intake), brushing teeth or taking oral medication, in order to avoid the saliva being contaminated with interfering substances. Up to 10 minutes before taking the sample, rinse mouth with water for 1-5 seconds.
 - 2 Immediately before collecting the saliva, wash and dry hands.
 - 3 With two fingers, remove the absorbent roll at the far end from the opened Salivette® Cortisol and
 - 4 place it in the mouth.
 - 5 The absorbent roll should be kept in the mouth (e.g. in the cheek) for 2 minutes without chewing. The amount of saliva collected should be at least 500 µl. If an extremely small amount of saliva is produced, leave the absorbent roll in the mouth longer.
 - 6 With two fingers, take the absorbent roll out of the mouth by the far end and
 - 7 place it immediately in the Salivette® Cortisol and seal with the blue cap.
For the patient: To return the saliva sample in the tightly closed Salivette® Cortisol, follow the doctor's instructions and use the secondary container provided by the doctor and the shipping box/bag. If it is not returned immediately, store this Salivette® Cortisol in the refrigerator.
- For doctors and laboratories:
- 8 Because of the onset of bacterial growth in saliva after a few hours at room temperature, it is recommended to either centrifuge the firmly closed Salivette® Cortisol within 4 hours after specimen collection at 1,000 x g for 2 minutes and analyse it or to store it immediately in the refrigerator until further processing.
 - 9 After centrifugation, take the insert including cap out of the conical bottom plastic tube.
 - 10 The pure saliva for further analysis is found in the conical bottom of the plastic tube above the residues that are located in the reservoir of the conical bottom.
 - 11 After analysis, dispose of the conical bottom plastic tube and the insert in a disposal container.

Disposal – for doctors and laboratories

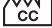
1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled Salivette® Cortisol tubes must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Literature

SARSTEDT Whitepaper <https://www.sarstedt.com/en/products/diagnostic/salivasputum/Literature/> "Comparing cortisol levels of saliva samples collected with the passive drool collection method with the Salivette® Cortisol]"

Kirschbaum, Hellhammer "Salivary Cortisol", Encyclopedia of STRESS 1st ed., vol. 3:379-383, 2000 (cited in 2nd ed. 405-409 2007; doi.org/10.1016/B978-012373947-6.00334-2)

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE marking
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Store away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product must be reported to the manufacturer and the competent national authority.

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SARSTEDT Whitepaper [https://www.sarstedt.com/en/products/diagnostic/salivasputum/Literature/ „Comparing cortisol levels of saliva samples collected with the passive drool collection method with the Salivette® Cortisol“](https://www.sarstedt.com/en/products/diagnostic/salivasputum/Literature/„Comparing_cortisol_levels_of_saliva_samples_collected_with_the_passive_drool_collection_method_with_the_Salivette®_Cortisol“)
 Kirschbaum, Hellhammer "Salivary Cortisol", Encyclopedia of STRESS 1st ed, vol 3:379-383, 2000 (cited in 2nd ed. 405-409 2007; doi.org/10.1016/B978-012373947-6.00334-2)

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